




Medical-Surgical and Step-Down Unit Personnel

PleuraFlow® Active Clearance System® Was In Use

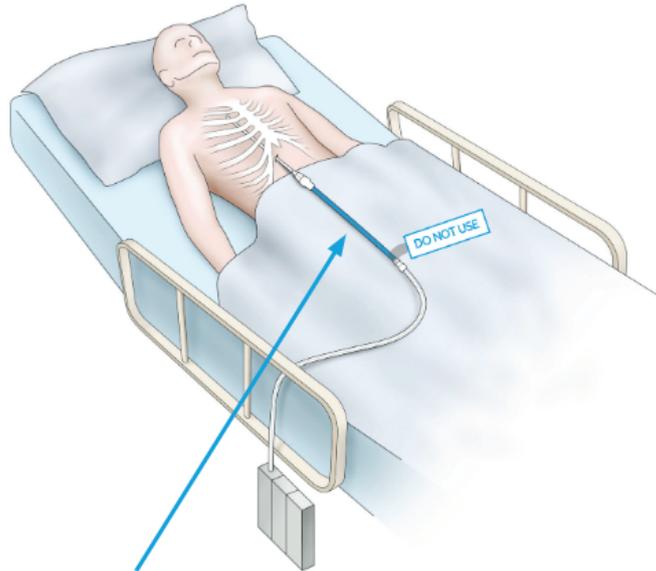

PleuraFlow Clearance Wire should be retracted in PleuraFlow Guide Tube, taped and labeled, “**DO NOT USE**”. If the Clearance Wire has not been retracted, taped and labeled, please contact your internal resource per hospital policy for instructions.



**The PleuraFlow Active Clearance System® is not MRI compatible.
Do not take patient to MRI with PleuraFlow System® in place.**




For Clinical Support call 1-844-257-3569 ext2
www.clearflow.com

DC105-C



Medical-Surgical and Step-Down Unit Personnel

PleuraFlow® Active Clearance System® Was In Use

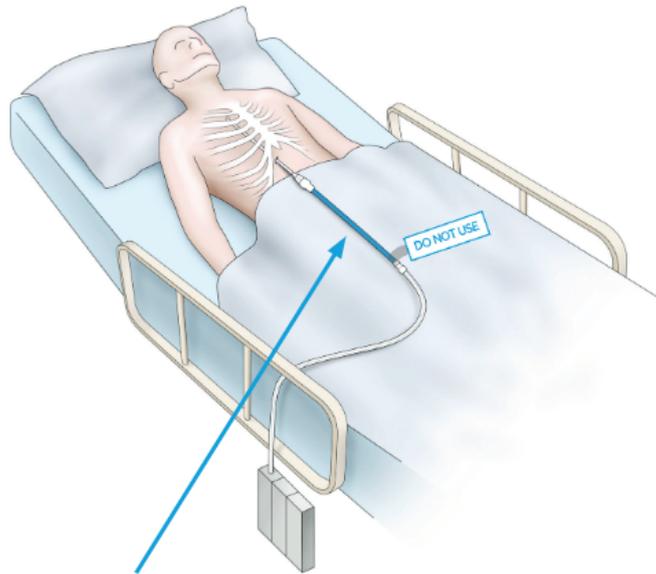

PleuraFlow Clearance Wire should be retracted in PleuraFlow Guide Tube, taped and labeled, **“DO NOT USE”**. If the Clearance Wire has not been retracted, taped and labeled, please contact your internal resource per hospital policy for instructions.



**The PleuraFlow Active Clearance System® is not MRI compatible.
Do not take patient to MRI with PleuraFlow System® in place.**



For Clinical Support call 1-844-257-3569 ext2
www.clearflow.com

DC105-C