

Additional Magnetic Strength

During placement or actuation, if additional magnetic strength is needed, depress and hold the Shuttle Guide Magnet Strength Button.

I

Chest Tube/Guide Tube Junction Not Flush

Ensure the chest tube/guide tube junction is flush and straight. If the chest tube was not cut straight on the "Cut" Line, try to straighten the cut and reattach the chest tube/guide tube junction.



Guide Tube Path Not Straight

Ensure the Chest Tube and Guide Tube are as straight and parallel to the patient's body as possible when actuating. Ensure the proximal end of the guide tube is not elevated when attempting to actuate.



Tips

If the internal and external magnets become uncoupled, advance or retract the Shuttle Guide over the internal magnet to recouple. Retaining elements set on the internal magnets will keep the internal magnets and wire from exiting the Guide Tube, thus encouraging recoupling of the magnets.

If decoupling occurs when the Clearance Wire and Loop are in proximity to the Parked Position, the Clearance Wire and Loop can continue to be used even if the Shuttle Guide does not fully click into the Parked Position.




If after several attempts the magnets remain uncoupled, the PleuraFlow Guide Tube may be disconnected from the PleuraFlow Chest Tube. The Chest Tube may then be connected to the drainage tubing and canister in the standard fashion.

If decoupling occurs when the Clearance Wire and Loop are in proximity to the parked position, continue using the device if:

- It is not possible to click the Shuttle Guide into the proximal barb housing to park the Clearance Wire and Loop in the proximal end of the Chest Tube, and;
- There is no resistance to wire movement inside the chest tube distally to the point of decoupling.




If a "Y" connection is utilized, additional drainage tubing may be used to compensate for length discrepancies.

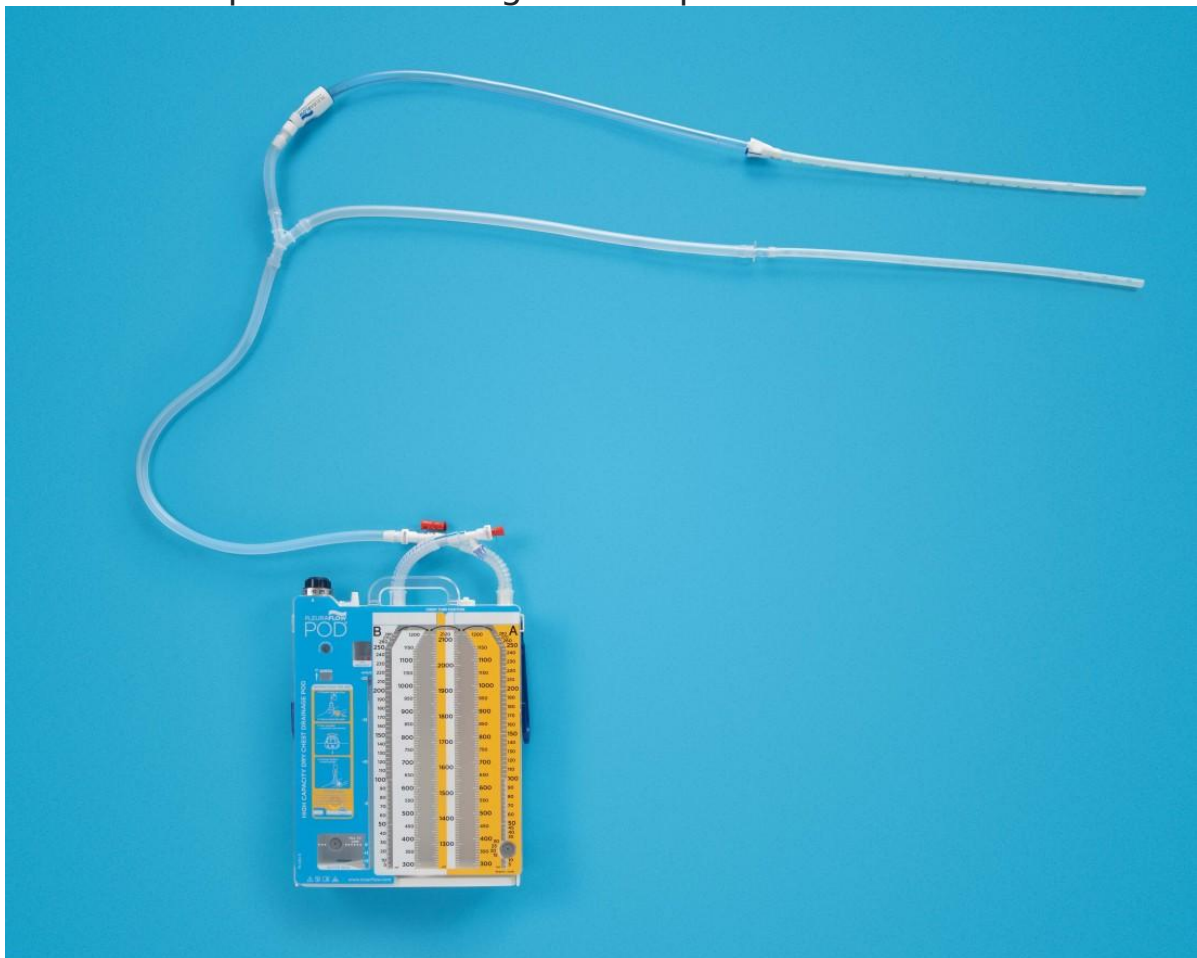


Withdrawing/Retraction: If decoupling persists during retraction, slide the shuttle guide over the internal magnet to recouple and return to the parked position. Communicate the location of decoupling to the ICU.

Advancing/Parking: If decoupling persists during advancement, slide the shuttle guide over the internal magnet to recouple and

retract the clearance wire out of the chest tube, leaving it in the guide tube. Notify the ICU of the position of the Shuttle Guide and communicate the location of decoupling.

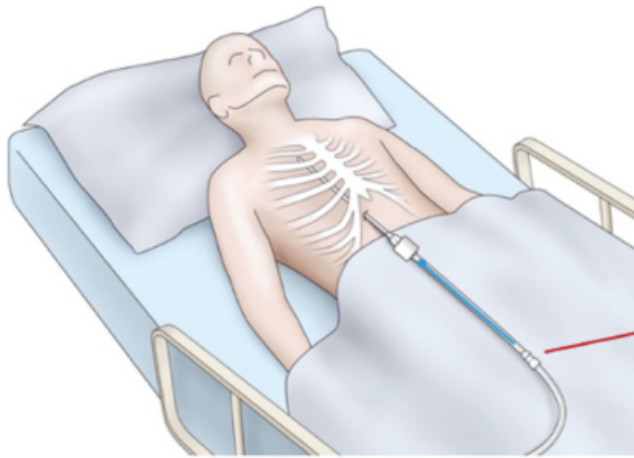

Distal End – Withdrawn Position

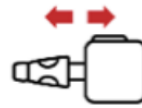

Farthest from Patient
Clearance Wire & Loop
are inside **Guide Tube**